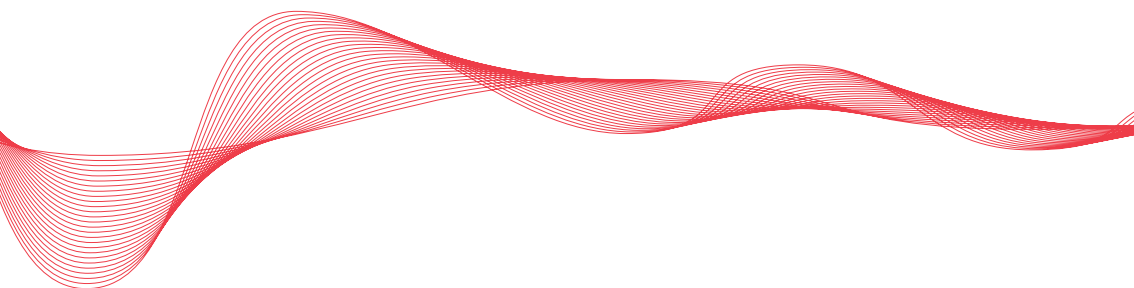
PROVOX[®] ActiValve[®]

Provox[®] Brush

Provox[®] Plug

Provox[®] ActiValve[®] Lubricant

Prescriber Information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and Trademarks

Provox® and Provox® ActiValve® are registered trademarks owned by Atos Medical AB, Sweden.

For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore; Fabricante; Fabricante; Tillverkare; Producent; Produsent; Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător; Proizvodač; Proizvođač; Κατασκευαστής; Προизводител; Üretici; Өндөрөгдөгч; Ἐργαστήριο; Ístehsalçi; Производител; Produsen; Pengilang; 製造業者; 제조사; 製造商; 製造商; 7737; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de fabrication; Data di fabbricazione; Fecha de fabricación; Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev; Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás időpontja; Datum výroby; Datum proizvodnje; Data produkcji; Data fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία κατασκευής; Дата на производство; Üretim tarihi; Ծանոթցման օրհոլդ; Արտադրման ամսաթիվ; Ístehsal tarixi; Дата изготовления; Tanggal produksi; Tarikh pembuatan; 製造日; 제조일자; 製造日期; 制造日期; 7737; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite d'utilisation; Data di scadenza; Fecha de caducidad; Data de validade; Används senast; Anvendes inden; Brukes innen; Viimeinen käyttöpäivä; Sĺđasti notkunardagur; Kőlblind kuni; Derĩguma termĩnš; Naudoti iki; Datum expirace; Felhasználhatóság dátuma; Datum najneskoršej spotreby; Rok uporabnosti; Data waźności; Data expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок на годност; Son kullanım tarihi; ջանցոծոս; Պիտանելիության ժամկետ; Son istifadə tarixi; Срок годности; Gunakan sebelum tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期; 使用截止日期; 使用截止日期; 7737; تاريخ انتهاء الصلاحية

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Provox® ActiValve® Voice Rehabilitation System

Stomal airway management

Emergency situations

It is important that emergency personnel knows that you are a neck breather. This will help them determining that oxygen and rescue breathing need to be administered to your stoma and not to your mouth or nose.

We recommend that you and your clinician download the guidelines for rescue breathing from

www.atosmedical.com/rescuebreathing

In the package of your Provox ActiValve you will find a bright yellow Emergency card and Provox ActiValve User Card. We recommend that you keep both the Emergency Card and Provox ActiValve User Card with you at all times.

Elective situations

If you need to undergo a procedure that requires intubation (putting a breathing tube in your windpipe), it is very important that the anesthesiologist and the doctor who is performing the procedure are aware that you are a neck breather and that you are using a voice prosthesis. It is important for them to understand that the voice prosthesis should stay in place. If it is removed, fluids from your mouth, food pipe, or stomach may get into your windpipe.

It is very important that the intubation tube is inserted and removed carefully so that the voice prosthesis does not dislodge or come out.

1. Glossary

Clinician - Medical professional or properly licensed speech and language therapist / pathologist or clinical specialist nurse who is trained in voice rehabilitation procedures.

HME - Heat and Moisture Exchanger. Device that retains the heat and moisture of exhaled air, which is returned to the air when you breathe in.

PE-Segment - Pharyngo-esophageal segment. The part of the food pipe where sound is created by vibration of tissue when using a voice prosthesis.

TE-puncture – Tracheoesophageal puncture. Small artificial opening created between the wind pipe and the food pipe in which the voice prosthesis is placed.

Tracheostoma - Breathing opening in the front of the neck, where the windpipe is connected to the skin (also called 'stoma').

Voice prosthesis - A one-way valve that is inserted into the TE puncture. The valve has a flange on either side that allows the valve to stay in place. Air can be redirected through the valve to the food pipe for speaking, while reducing the risk of food and liquids entering the wind pipe.

2. Descriptive information

2.1 Indications for use

Provox ActiValve is an unsterile indwelling voice prosthesis intended for antegrade insertion in a healed puncture for voice rehabilitation after total laryngectomy. The device is intended for patients who are experiencing early leakage with previous voice prostheses (device life less than 4-8 weeks). The device reduces the need for frequent replacements in a majority of users, but not in all.

2.2 CONTRAINDICATIONS

Provox ActiValve is NOT intended:

- for insertion in a freshly made puncture,
- to be in place during MRI-examination (Magnetic Resonance Imaging), or during Radiation Therapy.

2.3 Description of the device

Provox ActiValve Voice Prosthesis [Fig. 1] is an unsterile single use one-way valve that keeps a TE-puncture open for speech, while reducing the risk of fluids and food entering the windpipe. It has two retention flanges [A] and a blue ring [B] that adds stability and an even sealing surface for the valve flap [C]. The prosthesis is not a permanent implant and needs periodic replacement.

The device is made of medical grade silicone rubber and radio-opaque fluoroplastic. No part of the device is manufactured with natural rubber latex. The outer diameter is 7.5 mm (22.5Fr). Provox ActiValve is available in lengths 4.5, 6, 8, 10, and 12.5 mm and comes in different opening forces of the valve. The magnets [D] in the ring and valve flap determine the opening force (the magnets are not adjustable). Since the opening forces are not apparent upon simple visual inspection the device comes with Provox ActiValve User Cards that also provide important information about the voice prosthesis. If you have lost the user cards contact your clinician who can arrange for a replacement.

The following items in the package are intended for patient use and should be handed over to you by your clinician:

2 Provox Brushes,
1 Provox Plug,
1 Provox ActiValve Lubricant,
1 Emergency Card.
2 Provox ActiValve User Cards.

2.4 WARNINGS

Accidental swallowing of Provox ActiValve voice prosthesis may occur. If this occurs, contact your physician who will advise you further.

Accidental aspiration (i.e. the device falls into the windpipe) of Provox ActiValve voice prosthesis or other components of the Provox ActiValve voice rehabilitation system may occur. Immediate symptoms may include coughing, choking or wheezing. If this occurs seek immediate medical treatment. A foreign body in your airways may cause severe complications and has to be removed by a clinician.

Caution: You may aspirate or swallow the device without having immediate symptoms, or only mild coughing. If the device is not visible in the puncture contact the clinician immediately whether you have severe, mild, or no symptoms.

Re-use and re-processing may cause cross- contamination and damage to the device, which could cause patient harm.

Do not modify Provox ActiValve voice prosthesis or any of its accessories. This may increase the risk of aspiration or ingestion of the device.

2.5 PRECAUTIONS

Consult your clinician if:

- There is leakage through or around the prosthesis (coughing and/or change of mucus color).
- Speaking becomes difficult (higher effort and/or voice sounds more strained).
- There are signs of inflammation or tissue changes at the puncture region (pain, heat, swelling, traces of blood on the brush after brushing).
- You see changes in the appearance of the material of the voice prosthesis or the way it fits in the puncture.
- You frequently suffer from coughing during drinking, and cleaning the voice prosthesis does not resolve it. This may be an early sign of leakage through or around the voice prosthesis.
- You have bleeding or overgrowth of tissue around the device.
- You have persistent pain or discomfort in the region of Provox ActiValve.
- You experience chronic cough, trouble breathing, or blood in the mucus. These can be signs of a serious health condition that requires medical attention.

To reduce infection risk:

Make sure that your hands are thoroughly cleaned and dry before placing your hands in the area of your tracheostoma, and before cleaning your voice prosthesis and/or accessories.

Keep all devices clean that may enter or make any contact with your tracheostoma and voice prosthesis.

To reduce risk of product damage:

Only use genuine Provox accessories that are intended for use with Provox ActiValve for the handling and cleaning of your prosthesis. **Other devices may cause personal injury or damage to the voice prosthesis.**

Clean and lubricate the voice prosthesis as directed to help maintaining its functionality.

Only use Provox ActiValve Lubricant for lubrication of Provox ActiValve voice prosthesis. Other lubricants may damage the voice prosthesis.

To reduce risk of injury:

Always inform medical personnel about your Provox ActiValve and show them your Provox ActiValve User Card before undergoing any kind of medical treatment or examination. Also see section Contraindications.

Keep Provox ActiValve Lubricant and other accessories out of reach of children and pets that may be harmed by inhaling or swallowing them.

Do not try to change Provox ActiValve at home or by yourself. The device should only be removed and replaced by a healthcare professional.

3. Instructions for use

3.1 How to use Provox ActiValve for speaking

Some people have negative pressure in the pharynx/ esophagus during swallowing or inhalation. This can cause the valve to open unintentionally. The magnets in the valve increase the initial pressure needed to open the valve. Thereby unintentional opening is prevented.

As a consequence, you may feel that you need more force to begin speaking. Once the initial magnetic resistance is overcome Provox ActiValve is similar to using any other Provox voice prosthesis to speak.

3.2 Cleaning and Maintenance

In order for you to speak with your voice prosthesis, it must be clean so that air can pass through it. Regular cleaning of the prosthesis also helps to prolong device life.

Brush the prosthesis with Provox Brush twice a day and after each meal to remove mucus and food remnants from the prosthesis. Flushing the prosthesis with Provox Flush also helps to clean the device.

Once a day, after cleaning in the morning, also lubricate the inner lumen of the voice prosthesis with one drop of Provox ActiValve Lubricant using a Provox Brush. Before use, read the Instructions for Use that accompanies Provox ActiValve Lubricant.

Caution:

- Do not apply Provox ActiValve Lubricant directly from the bottle to the voice prosthesis, since this could deposit the liquid in the lungs [Fig. 2]. This or overuse of the lubricant in quantity or frequency can result in severe coughing, pneumonia or serious breathing problems.
- Do not enlarge the bottle opening of Provox ActiValve Lubricant.

3.3 Device lifetime

Depending on individual biological circumstances the device life varies, and it is not possible to predict the structural integrity of the device over a longer period of time. The material of the device will be affected by e.g. bacteria and yeast, and the structural integrity of the device will eventually deteriorate.

Adding certain food items such as buttermilk, probiotics and caffeinated soft drinks may reduce the occurrence of bacteria and yeasts on the voice prosthesis. A well- balanced diet is important, please discuss with your clinician if changes in your diet are recommended.

3.4 Accessories

Provox Brush / Provox Brush XL is used to clean the inside of the prosthesis or apply Provox ActiValve Lubricant.

Provox Flush is an additional cleaning device that allows flushing of the prosthesis.

Provox Plug is a First Aid tool that temporarily blocks leakage through the prosthesis.

Provox ActiValve Lubricant serves as lubricant for the valve surface inside Provox ActiValve.

3.5 Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

4. Adverse events/ Trouble shooting by symptom

Indications for replacement of Provox ActiValve voice prosthesis include leakage through the valve, blockage of the prosthesis, bacterial and candidal overgrowth leading to degradation of the materials and /or excessive pressure needed to obtain speech. Other reasons for replacement include medical indications such as problems with the puncture tract.

It is the responsibility of your clinician to inform you about risks and benefits of prosthetic voice rehabilitation and to instruct you what to do in case you may encounter adverse events. If you are not sure that you have completely understood all instructions, ask your clinician for further explanation.

Symptom: Coughing (triggered by drinking or eating) while the prosthesis is in place.

Most common reasons: Leakage through or around the prosthesis.

Measures:

1. Clean the prosthesis with Provox Brush and if possible also with Provox Flush. There may be food remnants preventing the valve from proper closure.

If leakage continues:

2. Observe the prosthesis while drinking. Try to identify if it leaks through the center or around the prosthesis.

- a) If you think it leaks through the center you can use Provox Vega Plug while you are eating and drinking to prevent leakage. Notify your clinician that you need a new prosthesis.
- b) If you think it leaks around, or if the Plug does not stop the leakage, notify your clinician that you may have leakage around your prosthesis.

Symptom: Speaking is sometimes not possible (valve does not open).

Probable cause: The valve lid sticks to the valve seat, preventing the valve from opening. This becomes more likely the longer the valve has not been in use (e.g. overnight).

Measures: Clean the voice prosthesis and lubricate it (see section Cleaning and Maintenance). If cleaning and lubricating does not help: Contact your clinician.

Symptom: Speaking becomes more and more difficult.

Most common reasons: Valve obstruction, tissue swelling in the throat (e.g. during radiotherapy).

Measures:

1. Clean the prosthesis with Provox Brush and/or Flush. There may be food remnants increasing the airflow resistance.
2. If that does not help: Contact your clinician immediately.

Symptoms: Pain, heat, swelling, and rash (together or alone) in the area of the prosthesis or stoma.

Most common reasons: Tissue inflammation and infections.

Measures: Contact your clinician immediately.

Symptoms: Traces of blood are seen on the brush after brushing.

Most common reasons: Tissue inflammation and infections at the esophageal side.

Measures: Contact your clinician immediately.

5. Additional information

5.1 Air travel

This device contains 2 magnets; size 1-1.5 mm, weight 0.15 – 0.2 gram. The magnets are unlikely to set off metal detectors such as those used in airports but in order to avoid incidents it could be advisable to inform the security personnel in advance of your voice prosthesis and show them your Provox ActiValve User Card.

5.2 Date of printing and user assistance information

The version number of this Instruction for use and manufacturer contact information is located on the back cover.

6. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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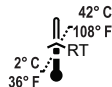
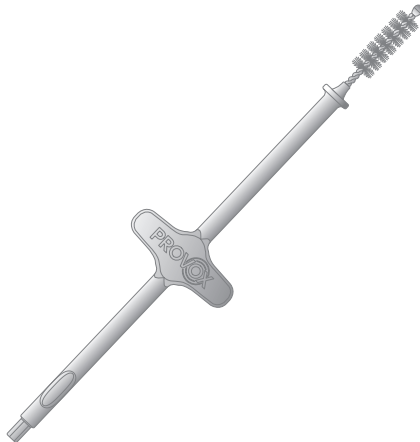
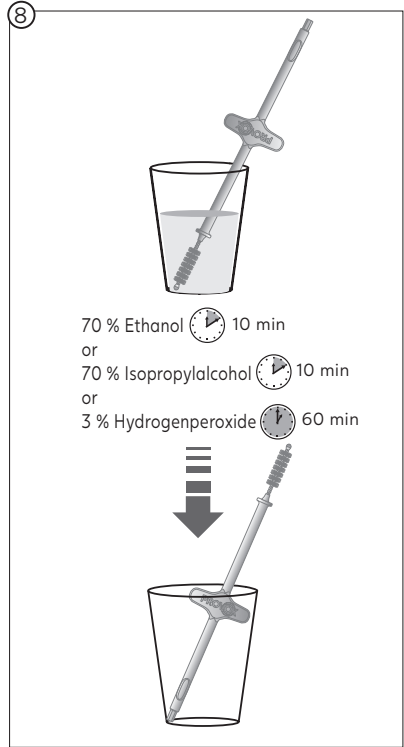
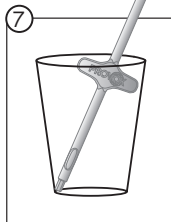
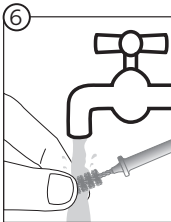
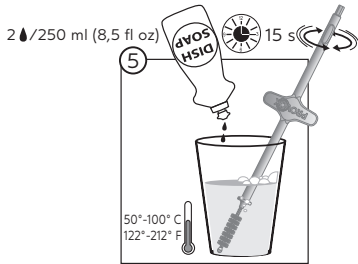
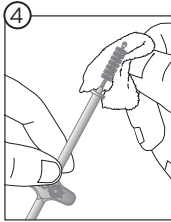
Intended Use

Provox Brush is a single patient use device intended for cleaning Provox Voice Prosthesis, for insertion of Provox Plug and Provox Vega Plug, for applying lubricant and anti-candida agents and for cleaning of fenestration holes on Provox LaryTubes. The product is intended for use by the patient.

Before use

Always inspect the Provox Brush (fig. 1) to make sure that it is not broken or worn out.

Before each use, make sure that

- the protective tip is not cracked or loose.
- the bristles do not appear worn or are loose.
- the wires are not bent or broken.
- the entire handle is not cracked or broken and that the safety wings have not been removed.

If any defects are found, the brush must be discarded.

Cleaning and disinfection

Clean the Provox Brush after each use according to fig. 4-7 using drinking water.

Disinfect the Provox Brush (fig. 8) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Do not use the device until it is completely dry

Note: Always clean and disinfect before use.

Precaution

- The device is intended for single patient use only. Reuse in another patient may cause cross contamination.
- Do not clean or disinfect by any other method as this might cause product damage and injury.

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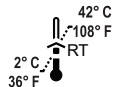
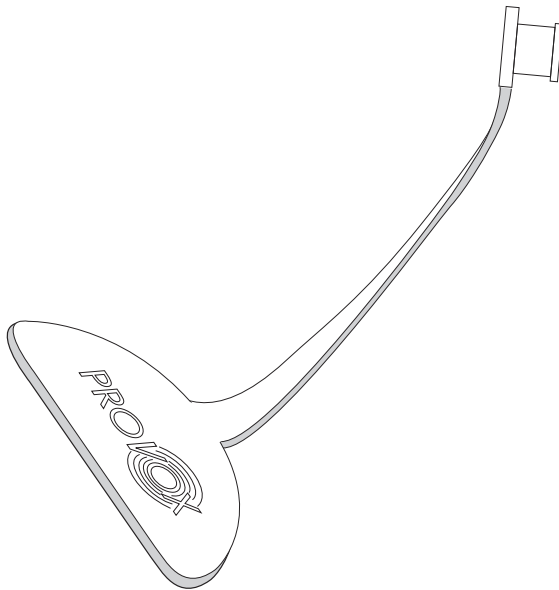
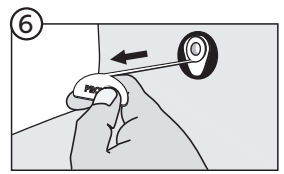
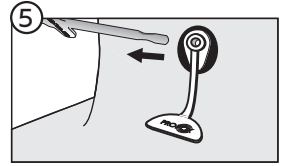
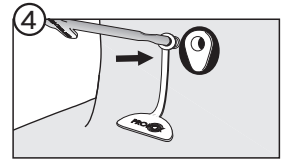
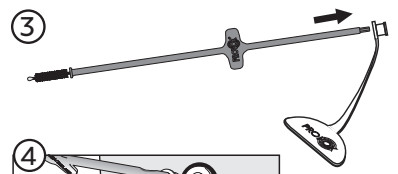
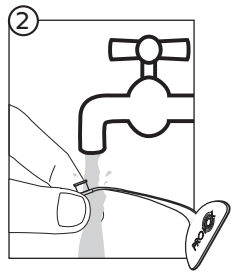
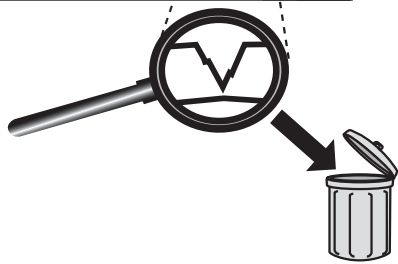
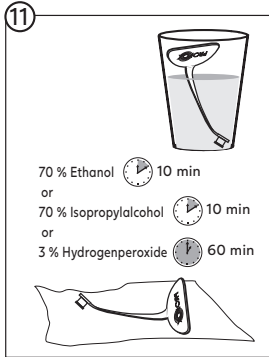
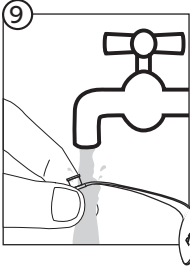
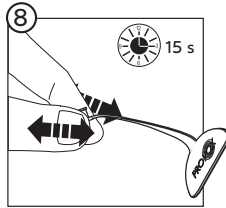
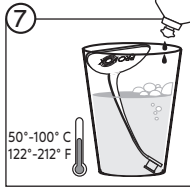
1. Descriptive Information

1.1 Intended use

Provox Plug is intended to seal the inner lumen of Provox Voice Prostheses, Provox2 Voice Prostheses and Provox ActiValve and therefore stop leakage of both air and fluids through the voice prosthesis.

1.2 Description of the device

Provox Plug is a first-aid tool for temporarily stopping leakage through the voice prosthesis. The device is inserted into the opening of Provox voice prosthesis. The medallion end can be taped to the skin if desired.

1.3 Precaution

Provox Plug is intended for single patient use.
Reuse in another patient may cause cross contamination.

2. Instructions for use

2.1 Operation instruction

Check device integrity prior to each use by pulling the strap see fig. 1.
For instructions on how to use the product, please see fig. 2-6.

2.2 Cleaning and disinfection

Clean Provox Plug after each use, or if it is changed during use according to Fig. 7-10.
Fig. 8, Rub the device between your fingers for 15 seconds.

DO NOT use any water other than drinking water to clean and rinse the device.
Disinfect Provox Plug at least once a day if it is removed (Fig. 11) with one of the following methods:

- Ethanol 70% for 10 minutes
 - Isopropylalcohol 70% for 10 minutes
 - Hydrogenperoxide 3% for 60 minutes
- CAUTION: DO NOT** clean or disinfect by any other method as this might cause product damage and injury.

If Provox Plug looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use.

A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect Provox Plug after use but also before use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

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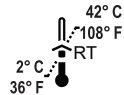
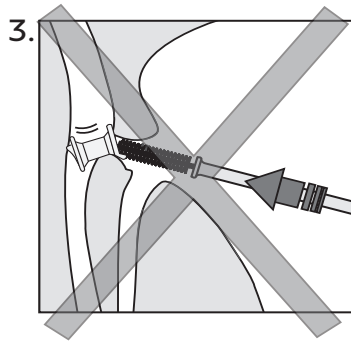
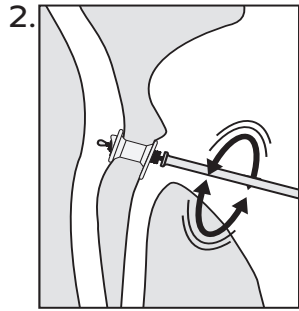
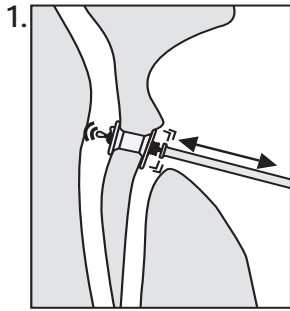
Indications for use

For use with Provox ActiValve only.

Lubricating the inner lumen of the Provox ActiValve prosthesis helps to prevent sticking of the valve that might otherwise occur e.g. after sleep.

Instructions for use

Once a day, after cleaning in the morning, lubricate the inner lumen of the Provox ActiValve prosthesis with one drop of Provox ActiValve Lubricant using a different Provox Brush. See illustrations.

Do not put more than one drop on the brush. Avoid touching the brush with the ActiValve Lubricant bottle when dispensing a drop. Be patient when squeezing the lubricant bottle. The lubricant bottle releases the thick silicone oil very slowly. If you get Provox ActiValve Lubricant on your skin wash it off with soap and water.

After use clean the brush according to the Instructions for use that accompanies the Provox Brush.

The ActiValve Lubricant can be used until expiration date or 8 months after opening the bottle, whichever occurs first. Register the date of opening on the bottle using a water resistant permanent marking pen.

Note: Avoid touching the stomal mucosa with the brush during application of the Provox ActiValve Lubricant.

Incidental ingestion of small amounts of Provox ActiValve Lubricant is harmless. Incidental aspiration of small amounts of Provox ActiValve Lubricant is also harmless, but may cause coughing.

No health problems have been reported regarding repeated aspiration of small amounts of silicone oil, but it is still better to be cautious and avoid it. Use only one drop of Provox ActiValve Lubricant each time and make sure that any superfluous silicone oil is removed before using the brush (no dripping brush).

Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

CAUTION:

- Overuse of the lubricant in quantity or frequency can result in severe coughing, pneumonia or serious breathing problems.
- Do not apply the Provox ActiValve Lubricant directly from the bottle to the prosthesis since this could deposit the liquid in the lungs and cause severe coughing, pneumonia or serious breathing problems.
- Do not enlarge the bottle opening of the Provox ActiValve Lubricant.
- Keep the Provox ActiValve Lubricant out of reach of children and pets that may be harmed by inhaling or swallowing it.

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© Atos Medical AB, Sweden.