


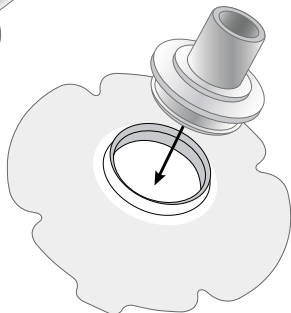
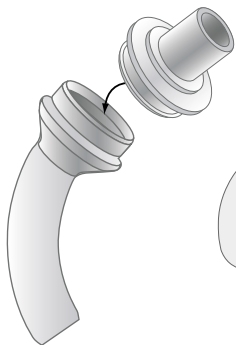
PROVOX[®] BasePlate Adaptor Life

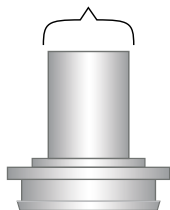
ISO 15 mm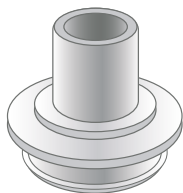


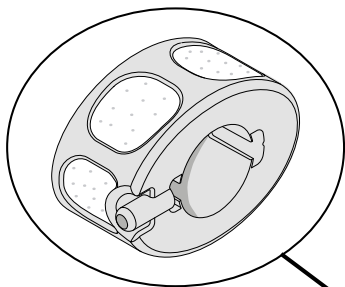

TrachPhone[®]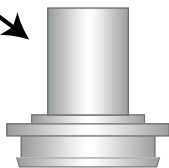

2  /250 ml (8,5 fl oz)

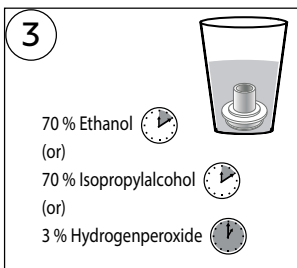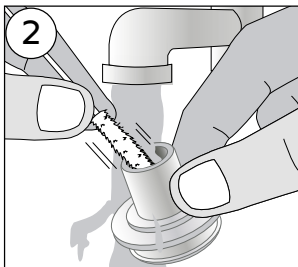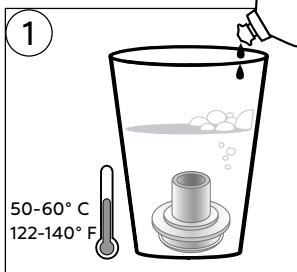Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patent and trademarks

Provox[®] is a registered trademark owned by Atos Medical AB, Sweden.

Provox[®] Life[™] is a trademark of Atos Medical AB.

For information about protective rights (e.g. patents), please refer to our webpage www.atosmedical.com/patents.

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore;
 Fabricante; Fabricante; Tillverkare; Producent; Producent;
 Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas;
 Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător;
 Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici;
 მწარმოებელი; Ἀρτοποιητής; İstehsalçı; Производитель;
 Produzen; Pengilang; 製造業者; 제조사; 製造商; 制造商;
 מוצר; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de
 fabrication; Data di fabbricazione; Fecha de fabricación; Data de
 fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato
 ;Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev;
 Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás
 időpontja; Dátum výroby; Datum proizvodnje; Data produkcji; Data
 fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία
 κατασκευής; Дата на производство; Üretim tarihi; წარმოების
 თარიღი; Ἀρτοποιημάτων ἀντιστοιχία; İstehsal tarixi; Дата
 изготовления; Tanggal produksi; Tarikh pembuatan; 製造日;
 제조일자; 製造日期; 制造日期; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite
 d'utilisation; Data di scadenza; Fecha de caducidad; Data de vali-
 dade; Används senast; Anvendes inden; Brukes innen; Viimeinen
 käyttöpäivä; Síðasti notkunardagur; Kõlblik kuni; Derīguma
 termiņš; Naudoti iki; Datum expirace; Felhasználatóság dátuma;
 Dátum najneskoršej spotreby; Rok uporabnosti; Data ważności; Data
 expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок на
 годност; Son kullanım tarihi; ვარგისობა; Ἡμερομηνία ἰσοπέρας
 διατήρησης; Son istifadə tarixi; Срок годности; Gunakan sebelum
 tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期;
 使用截止日期; تاريخ انتهاء الصلاحية

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Intended use

Provox Life™ BasePlate Adaptor is an accessory that allows attaching medical devices, e.g. an HME, with an ISO 15 mm standard connector to a Provox Life™ attachment.

Contraindications

Shall not be used for mechanical ventilation.

Description of the device

Provox Life™ BasePlate Adaptor is an accessory that allows attaching and detaching medical devices, e.g. an HME, with an ISO 15 mm standard connector to a Provox Life™ attachment.

WARNINGS

- Unconscious patients must be constantly monitored. Attached devices may inadvertently slide off the adaptor, or the baseplate may detach from the tracheostoma.
- Make sure the adaptor is free from oil or any other lubricant before and during use. Lubrication may lead to detachment of devices from the adaptor, or of the adaptor from the baseplate.

PRECAUTIONS

- Read and follow the instructions of the medical device to be used with the adaptor.
- Never use force while attaching or detaching devices or the adaptor itself.
- Do NOT sterilize! The device is for single patient use only.
- Do not use if the device shows any signs of cracking or structural damage.

Cleaning and maintenance

Clean Provox Life™ BasePlate Adaptor daily as instructed in figure 1-2. Disinfect Provox Life™ BasePlate Adaptor with either hydrogenperoxide 3% for 60 minutes, ethanol 70% for 10 minutes or isopropylalcohol 70% for 10 minutes at least once a day (figure 3).

CAUTION: Do not use the device until it is completely dry. Inhalation of disinfectant fumes can cause severe coughing and irritation of the airways.

If Provox Life™ BasePlate Adaptor looks dirty with risk of contamination e.g. by contact with a pet, dropping on the floor or other gross contamination, or if someone has a respiratory infection, Provox Life™ BasePlate Adaptor should be both cleaned and disinfected prior to use.

During hospitalization it is important to clean and disinfect Provox Life™ BasePlate Adaptor immediately after use and again just before use due to an increased risk of infection and device contamination.

Use sterile water for rinsing, rather than drinking water.

Device lifetime

Provox Life™ BasePlate Adaptor may be used for a maximum period of 3 months. If the device shows any signs of damage it shall be replaced earlier.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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